

# VOYAGER


CONGRATULATIONS! YOU HAVE PURCHASED THE FINEST INSULIN SYRINGE DISPOSAL SYSTEM AVAILABLE  
 YOU AND YOUR PHARMACIST HAVE TAKEN A POSITIVE STEP TOWARD A CLEANER AND  
 SAFER ENVIRONMENT FOR EVERYONE

When filled, dispose in accordance with all state and local regulations.

- USE ONLY AS DIRECTED • FOR USED INSULIN SYRINGE NEEDLES ONLY
- CONTAINS BIOHAZARD / INFECTIOUS WASTE


To reduce the possibility of needle stick injury, handle with care during use, transportation and disposal. This unit is puncture-resistant, not puncture proof.

## STEP 1

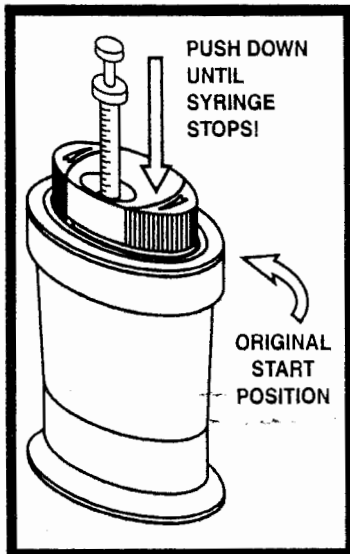


1. Holding the syringe barrel near the top, insert needle/syringe vertically into the center hole at top of unit until it stops (see illustrations on back).

## STEP 2

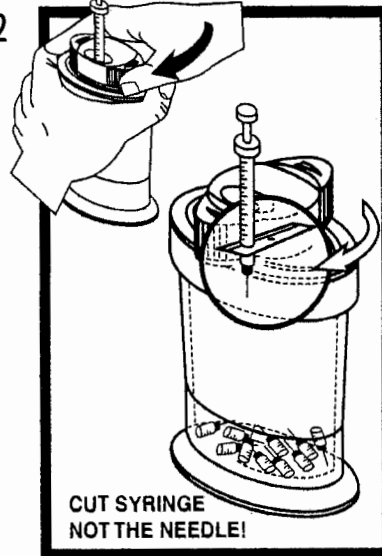

2. Hold **VOYAGER** securely with one hand above label area. With other hand, use steady pressure and **SLOWLY** twist the outside of the top to the right in the direction of the arrows (approx. 1/4 of a turn). Return the outside of the top back to the "Original Start Position".

## STEP 3

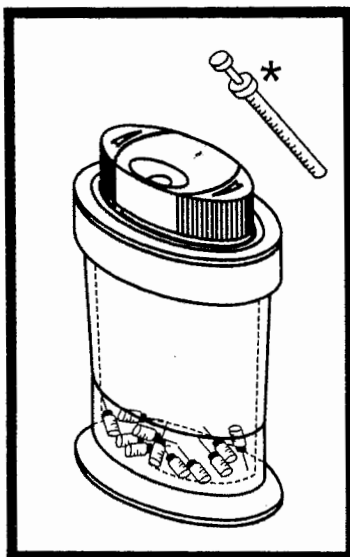

3. Needle removal process is now complete. For safe transport and temporary closure, remove the cap from the bottom of the container and replace on top.

\* Dispose the remaining syringe barrel in household waste.

## STEP 4



4. After contents reach the **MAXIMUM FILL LINE** - **DISCONTINUE USE**. Remove cap from the bottom of container and snap firmly onto the top of the **VOYAGER**.

# VOYAGER

DIABETIC NEEDLE DISPOSAL  
ACCEPTS MOST LANCETS

Sharps Collection Container

- THIS UNIT ONLY ACCEPTS 1/3cc, 1/2cc & 1cc INSULIN SYRINGES
- PLACE REMOVABLE CAP ONTO TOP WHEN TRAVELING OR FOR TEMPORARY STORAGE
- CAP CAN ALSO BE USED AS A STAND

(1cc SYRINGE)



Insert syringe as shown before operating


(1/2 cc SYRINGE)



(1/3 cc SYRINGE)



(Lancet Disposal)

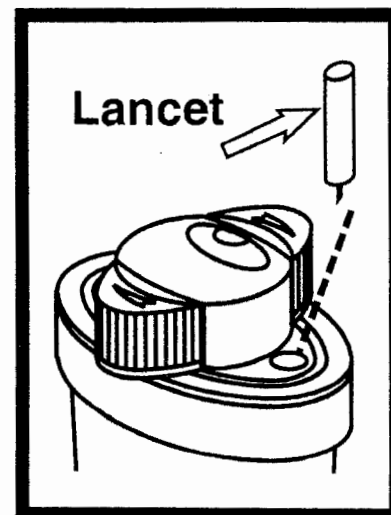

\*Note: Insert syringe to specified number on syringe barrel or Until It Stops (See Illustrations) or damage will occur to cutting blade.